


TICKERWORKS

COMPOUNDING TECHNOLOGIES

TICKER PEN FILLING INSTRUCTIONS FOR PHARMACISTS & TECHNICIANS*


*Please read all information, specifications and instructions carefully before dispensing in the Ticker.

STEP 1

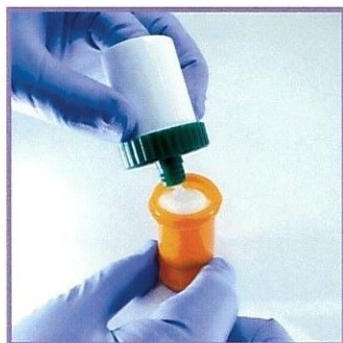


Use a spatula, EMP Jar, or a suitable filling device to fill the desired amount from the bottom to the top of the Chamber.

STEP 2

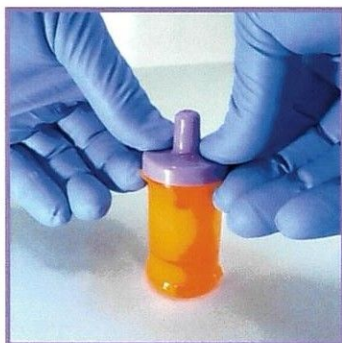


Snap the Dispenser Lid onto the Chamber to secure the ingredients inside of the Chamber-Complex.

STEP 3

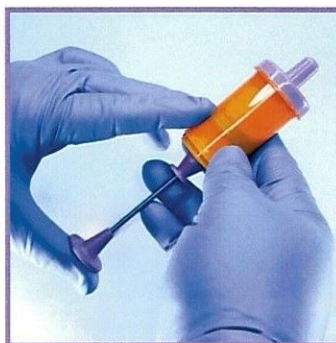


Prime the TICKER-X PEN by inserting Super Primer through the bottom of Chamber Complex

STEP 4

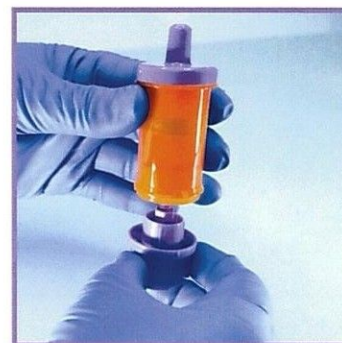


Add the Rotator Base/Screw Complex to the Chamber of the TICKER-X PEN to meterized the unit.

STEP 5

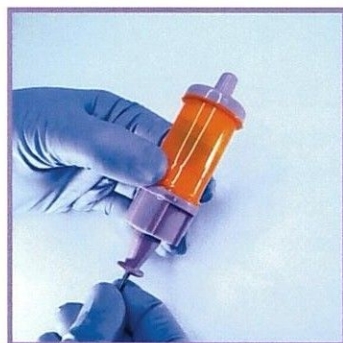


If additional priming is needed, use Super Primer, or rotate the base to the right (counter-clockwise) until the unit is primed.

STEP 6

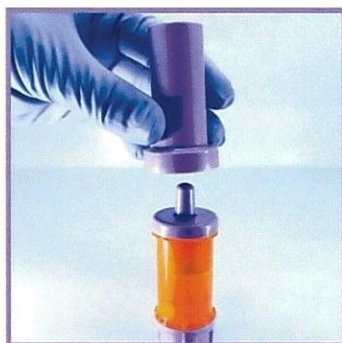


Insert Safety Cap over the TICKER-X PEN to Dispenser Lid.

STEP 7

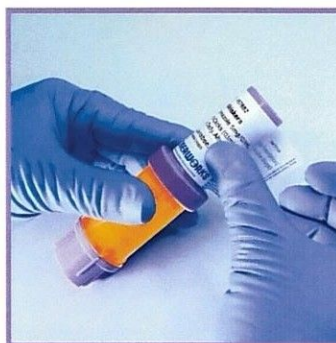

Label TICKER-X PEN and further package if necessary.

STEP 8

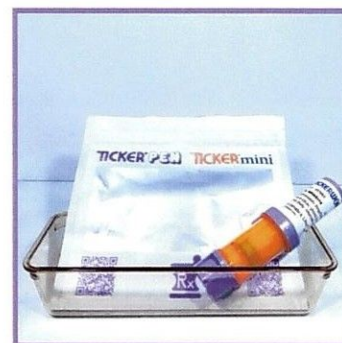

Give to pharmacist to check and prepare for dispensation.